GEBRAUCHSANLEITUNG
USER'S MANUAL

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PINO

PINO_2021_DE-EN

08-24

REF Artikel /
Article
26600 / 26650 / 26680

KOMPONENTENTEN

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CONTENTS

PINO components	2	Pedals	7	Riding your tandem	19
Description	4	Flammability	8	Correct tire pressure	19
What is the PINO?	4	Safety harness	8	Shifting gears	19
Intended use	4	Riding at night	8	Braking	19
Contraindications	4	Technical condition	8	Carrying cargo	20
Who can ride in the front seat?	4	Adjusting the frame length	8	Parking the PINO	20
Adaptability to special needs	4	Loading the Pino	8	Riding with children	20
Reuse	4	Serial number and bike coding	8	Removing the seat	21
Protection against environmental influences	5	Further information	8	Transporting the PINO	22
Cleaning and disinfection	5	General information	9	Technical data	23
Applicable standards and guidelines	5	Torque specifications	9	Maintenance and repairs	24
Disposal instructions	5	Quick-release levers	9	Service intervals	24
Safety information	6	Adjusting your tandem	10	Checking wheels for lateral/radial true	25
Read and observe the user's manual	6	Checking the seat position of the rear rider	10	Checking the tires	25
Steps motor option	6	Saddle adjustments	11	Checking the disc brakes	25
Intended use	6	Handlebar adjustments	12	Checking and adjusting the	26
Before your first ride	6	Adjusting the handlebars	13	derailleur	26
Observing traffic regulations	6	Adjusting the handlebar grips	14	Checking the chain tubes	26
Avoiding muscle strain	7	Checking the seat position of the front rider	15	Checking the chain	26
Braking	7	Adjusting the frame length using the front boom	16	Cleaning and oiling the chain	26
Suitable clothing and helmet	7	Adjusting the frame length using the middle frame	17	Rohloff Speedhub rear wheel disassembly	27
Risk of injury from the gear system	7	Adjusting the seat angle	18	Warranty	28
Pedaling around curves	7			Service log	31

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What is the PINO?

The PINO is a two-wheeled semi-recumbent tandem for two adult riders or for one adult rider with one child rider. The tandem can be adjusted to the size of the riders, making it an ideal family bike. Furthermore, it has been approved as a medical device under the European Directive (2017/745) MDR (Medical Device Directive). The tandem is propelled by leg-muscle power on the pedals, which can be equipped with accessories to prevent the feet from slipping off. The power is transmitted to the rear wheel via a derailleurs system or optionally a 14-speed Rohloff hub. The padded seats have a cushioning effect, thanks to their design and materials, and offer maximum comfort. The PINO's braking system consists of two independently operated disc brakes.

Intended use

The tandem is designed for use on paved roads and paths. The use of the PINO off-road or for competition is strictly forbidden. The tandem should not be ridden in traffic until it has been equipped in accordance with the applicable traffic regulations of the country in which it is being used.

Who can ride in the front seat?

The PINO is designed for people with neuromuscular conditions, such as cerebral palsy or muscular dystrophy, who, owing to their disability, are unable to use conventional bicycles or vehicles, even with support wheels. The tandem is also suitable for riders with coordination difficulties, balance disorders, restricted movement, e.g. associated with spasticity, paraplegia, dysmelia, amputations etc.

Contraindications

The rear rider who steers the PINO must be a healthy individual with no physical, mental, or visual impairments. No contraindications are known for the front-seat rider.

Adaptability to special needs

Special accessories are available for adapting the PINO to riders with various disabilities. Available accessories include Children's Cranksets, Crank Shorteners, Special Pedals with and without Calf Support, Pendulum Pedals, Harness Systems with lap and shoulder restraints, and Holders for Walking Sticks.

Reuse

Under normal conditions of use, the PINO can be reused by a new owner after inspection by an authorized dealer.

Protection against environmental influences

The PINO is made with corrosion-resistant materials. Coated and anodized aluminum frame parts, plastics and stainless steel are used.

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Cleaning and disinfection

The hand grips and seat can be cleaned and disinfected using common household cleaning and disinfecting agents.

Applicable standards and guidelines

In the design and manufacturing of the PINO, the following guidelines and standards were considered: European Directive (2017/745) MDR (Medical Device Directive).

Disposal instructions

At the end of its useful life, the product should be disposed of properly, in accordance with the local waste disposal regulations (i.e. not as household waste).

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Read and observe the user's manual

Please read this user's manual carefully and familiarize yourself with the construction and function of the vehicle in practice. Owing to the fact that both riders are active in operating the tandem, it is essential that both riders read this user's manual and learn about all of the bicycle's functions. If the front rider is a child, read the information together and explain the information you have read. Make sure that the child has understood all safety-related functions.

Period of use

With proper use, taking into account the instructions for use and depending on the frequency of use, the expected useful life is up to eight years.

Steps motor option

The frame of the PINO has been tested in combination with the optional Shimano Steps pedelec system and meets the requirements for electrically power assisted cycles (EPACs) in accordance with the European standard EN 15194. If you have purchased a HASE bike with the Shimano STEPS pedelec system, it will include a brief description with the link to a user's manual from the manufacturer Shimano, which includes all important information on the safe use of the Shimano STEPS pedelec system. The use of digital instead of printed manuals conserves resources and protects the environment.

Intended use

The PINO is a semi-recumbent tandem, which is a combination of an upright and a recumbent bicycle. It is intended for use on paved roads and paths. The tandem should not be used in traffic until it has been equipped with the accessories required by the applicable traffic regulations of the country and state in which it is being used.

Before your first ride

Your authorized retailer has assembled your PINO and adjusted it to the specified size requirements. Please refer to the section "Adjusting your tandem" if you need to adjust the PINO to fit another rider. Prior to your first ride, be sure to take some time to familiarize yourself with the vehicle. If the front rider is a child, be sure to explain all important functions and discuss potential risks. We recommend familiarizing yourself with the vehicle's handling in a quiet, open area away from traffic before riding on public roads. Owing to the unique construction, there are significant differences in handling in comparison with a standard bicycle or tandem. The rear and front riders must take some time to practice handling the tandem cooperatively. This is especially true for cornering, when the riders must shift the center of gravity.

The maintenance measures required before each ride are to be performed by the user (individuals with no physical, mental, or visual impairments).

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Observing traffic regulations

If you plan on adding equipment or making changes to the lighting system, be sure to study the applicable regulations before you begin or consult your authorized dealer and have the work performed by a qualified bike mechanic. Road traffic regulations can vary from one state or country to the next. It is important to choose a lighting system, reflectors, and a bell in accordance with the applicable regulations of the state and country in which the tandem will be used. We offer accessories that comply with the German road traffic regulations (StVZO).

Avoiding muscle strain

The muscle groups used by the front rider, who pedals in a recumbent position, differ from those used when riding an upright bicycle. If this rider has not had much experience with riding in a recumbent position, then we recommend starting with shorter, easy tours in order to avoid excessive muscle strain.

Braking

Your PINO is equipped with especially powerful brakes designed for use with tandems. Tandem brakes differ from the standard systems in several ways. When making repairs or replacing parts, be sure to use

- only braking systems specially designed and approved by HASE BIKES for the PINO
- and original brake discs with a diameter of 180-203 mm in the back and 203 mm in the front.

The disc brakes of the PINO are very powerful and offer good modulation. However, sudden hard braking can nevertheless cause your wheels to lock. Before your first ride in public traffic, be sure to familiarize yourself with the braking force by trying a few braking trials in an open area away from motorized vehicles. Furthermore, a new braking system must be “broken in” before being used in road traffic. This breaking-in process requires 25-30 full braking operations.

Suitable clothing and helmet

PINO riders should never wear loose clothing that could get caught in the wheels or gears. Beware of scarves, skirts, and any strings, cords or straps that hang down from clothing. Wear brightly colored clothes (ideally signal colors) in order to increase visibility for other road users. A bicycle helmet can prevent head injuries in the case of accidents.

Risk of injury from the gear system

PINO riders in the front seat should never touch the gear system when the tandem is in motion, as doing so could cause dangerous injuries.

Pedaling around curves

Depending on the length adjustment of the tandem, the heels of the front rider could bump against the front wheel in tight curves. Therefore, the front rider should not pedal when making tight cornering maneuvers. For smaller riders, this risk could be reduced by using shorter crank arms (available as accessories).

Pedals

In a recumbent position (front rider), it is especially easy for the feet to slip from the pedals. Wear shoes with slip-resistant soles. Be sure to maintain sufficient pressure on the pedals. We recommend using clipless pedals with the corresponding shoes in order to ensure that

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the rider's feet are secured. For children, the Special Pedals with straps, which are available as accessories, are especially suitable. Pedals that hold the feet securely in place are particularly important when the front rider is a child because children sometimes fall asleep while riding.

Flammability

The upholstery material of the seat cover is not resistant to ignition sources as specified in EN 1021-2. Therefore, be sure to keep the seat away from open flames, heat sources, etc.

Safety harness

Secure children in the front seat using the five-point Harness System (accessory) in order to prevent injury in the case of abrupt braking or accidents.

Riding at night

Always use lights when riding at night or in conditions of low visibility, such as dusk and dawn.

Technical condition

Before each trip, be sure to check the condition of the bicycle for possible faults or damage. Take your PINO to an authorized dealer for regular servicing. The recommended period of time between maintenance checks can be found in the section "Service intervals." Use only replacement parts and accessories that are recommended by HASE BIKES. Consult your authorized dealer for more information.

Adjusting the frame length

The length of the PINO frame can be adjusted quickly and easily. This telescoping mechanism also makes it possible to reduce the length for transport. After the length has been adjusted, the loosened screws must be retightened.

Loading the PINO

Never exceed the max. permissible load (combined weight of riders and cargo) of 225 kg (496 lb). All cargo must be transported using the rear rack or a compatible HASE BIKES accessory: e.g. Lowrider Rack, Porter Rack or Porter Bag. HASE BIKES only allows the use of two-wheel trailers. The use of a single-wheel trailer can cause damage to the frame due to the low resistance to tipping. Ensure that the weight of the cargo is distributed evenly.

Serial number and bike coding

The frame number is laser-etched onto the middle frame under the front seat. This is the only place where a reference number for bike coding can be attached. Bike coding gives owners a better chance of getting their bicycle back if it is lost or stolen.

Further information

If you have any problems or questions about your therapeutic bike or trike, please contact your authorized dealer or HASE BIKES for assistance.

Any serious incident* that occurs in connection with the product must be reported to the manufacturer and the responsible competent authorities of the member state in which the user and/or patient resides.

*A "serious incident" is any incident that had, could have had, or could have one of the following consequences, either directly or indirectly: a) the death of a patient, user, or other person, b) the serious temporary or permanent deterioration of the health condition of a patient, user, or other person, c) a serious risk to public health.

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TORQUE SPECIFICATIONS

Use a torque wrench for all bolts. If not specified in the instructions, the table gives the guide values for all screws of 8.8 quality.

Bolt	Tightening torque	Bolt	Tightening torque
M3	1.2 Nm (0.9 ft-lb)	M6	9.0 Nm (6.6 ft-lb)
M4	2.9 Nm (2.1 ft-lb)	M8	24.0 Nm (17.7 ft-lb)
M5	5.5 Nm (4 ft-lb)	M10	46.0 Nm (34 ft-lb)
M8-Frame-clamping	14 Nm (10.33 ft-lbs)		
M8-Steeringrod-clamping	14 Nm (10.33 ft-lbs)		
Through axle front wheel	15 Nm (11.06 ft-lbs)		
Through axle rear wheel	15 Nm (11.06 ft-lbs)		
Rohloff axle bolts	15 Nm (11.06 ft-lbs)		

QUICK-RELEASE LEVERS

Opening

Swing the lever **(01)** 180° up and over to the open position (the word "OPEN" will be visible). In order to open the quick-release mechanism even further, brace the tension-adjusting nut while turning the quick-release counterclockwise.

Closing

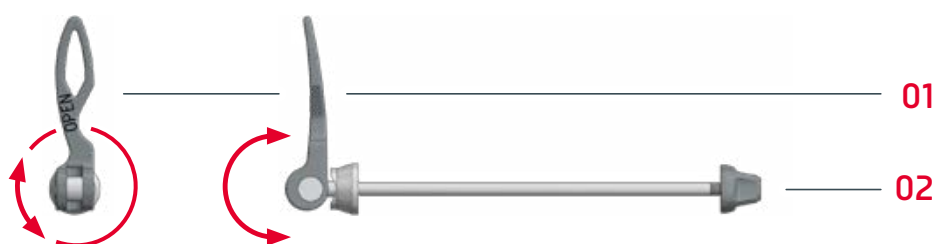
Use the tension-adjusting nut **(02)** to adjust the tension of the quick-release mechanism.

PLEASE NOTE! The adjustment is correct when you can fully close the lever **(01)** but with some effort (requiring pressure from the palm of your hand).

Swing the lever **(01)** 180° up and over to the closed position (the word "CLOSE" will be visible).

PLEASE NOTE! If the quick-release lever **(01)** can be closed using only finger pressure, then the tension-adjusting nut will need to be further tightened.

CAUTION! Ensure that all quick-release mechanisms are tightly secured before using the tandem.

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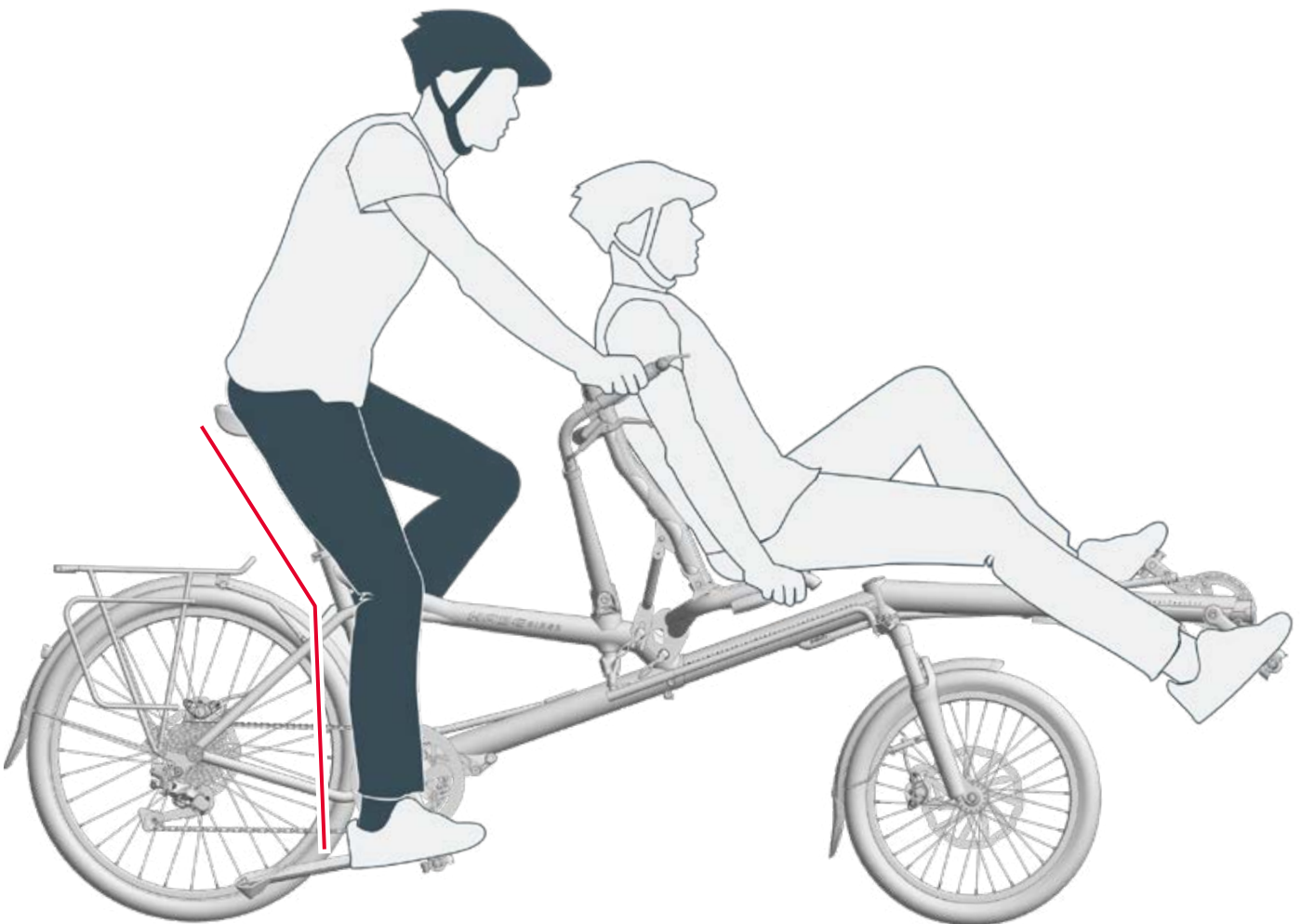
The PINO can be adjusted for use by rear riders from approx. 1.50 to 2.00 m (4'11" to 6'7") in height and front riders from 1.00 to 2.00 m (3'3" to 6'7") in height.

i **PLEASE NOTE!** Front riders who are less than 1.50 m (4'11") in height will require shorter crank arms.

CHECKING THE SEAT POSITION OF THE REAR RIDER

Sit on the saddle of the PINO with your feet on the pedals and with the balls of your feet centered over the pedal axles. Your leg should be almost fully extended at the bottom of the pedal rotation.

i **PLEASE NOTE!** Consult your authorized dealer for assistance.

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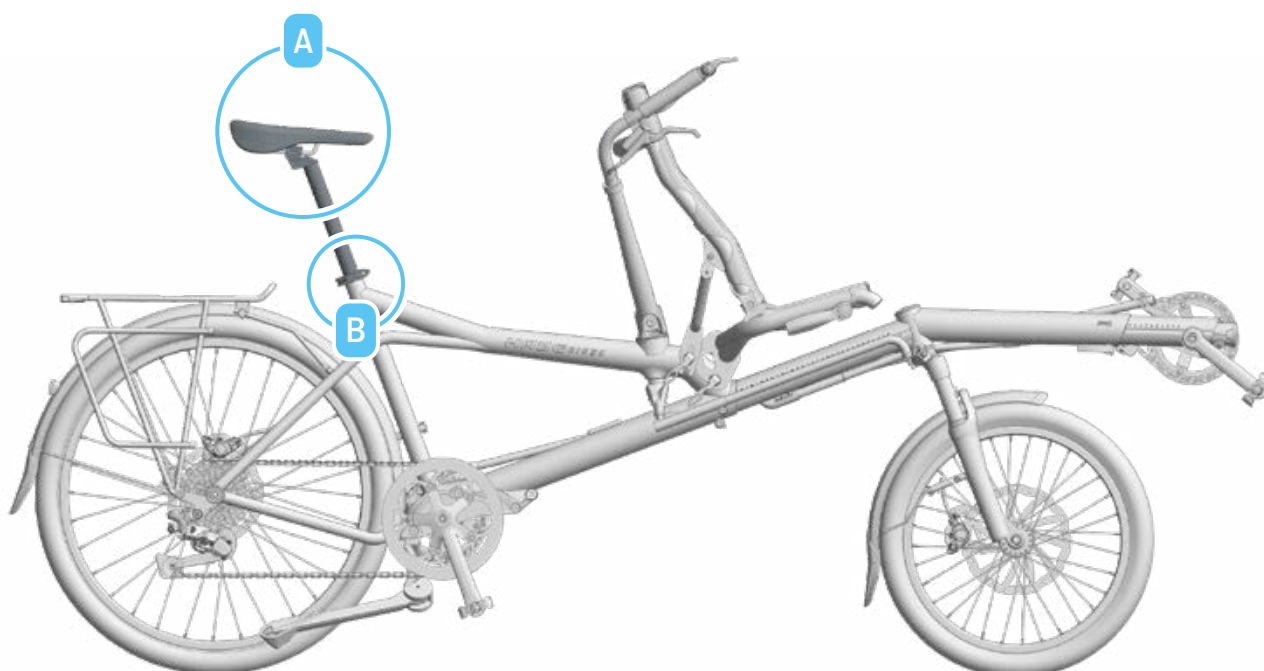
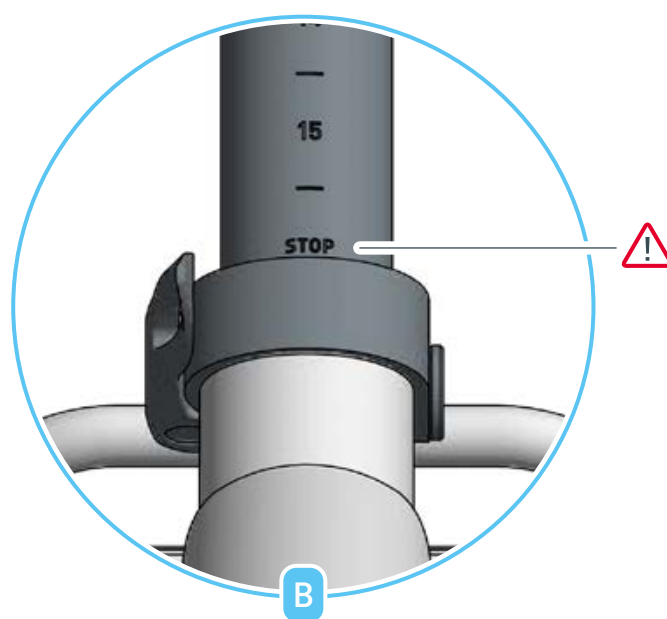
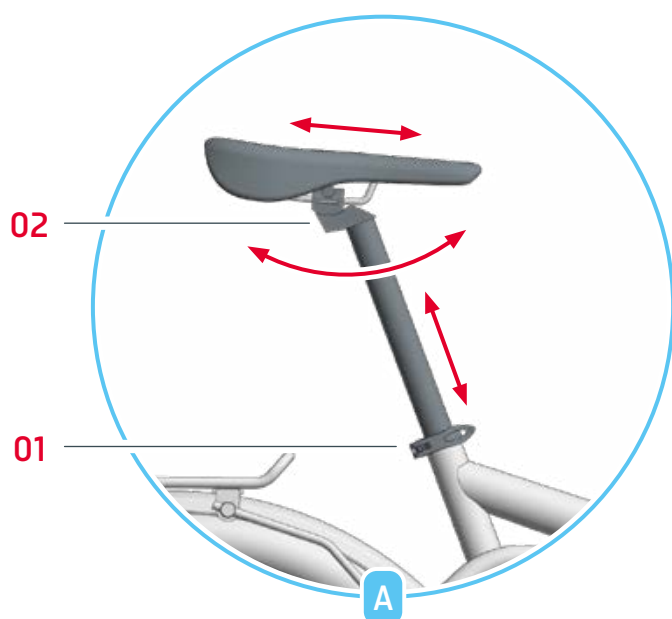
SADDLE ADJUSTMENTS

1. To adjust the **height**, open the quick-release lever **[01]**, and either pull the seatpost upwards or push it further into the seat tube.

 **CAUTION!** Never ride your bike with the seatpost drawn out beyond the "STOP" mark. **(B)**

2. Retighten the quick-release mechanism **[01]**.
3. In order to adjust the **saddle position**, loosen the bolt **[02]** under the saddle. The **angle** of the saddle can be changed, and it can be moved **forward** or **backward**.
4. Retighten the bolt **[02]**.

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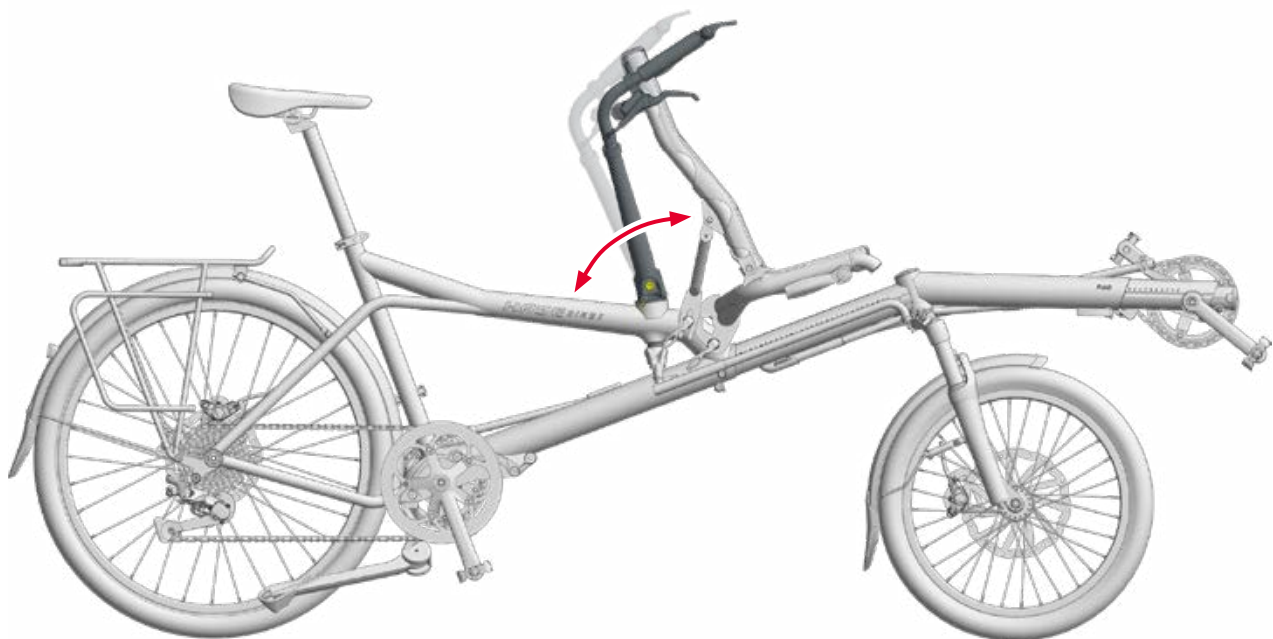
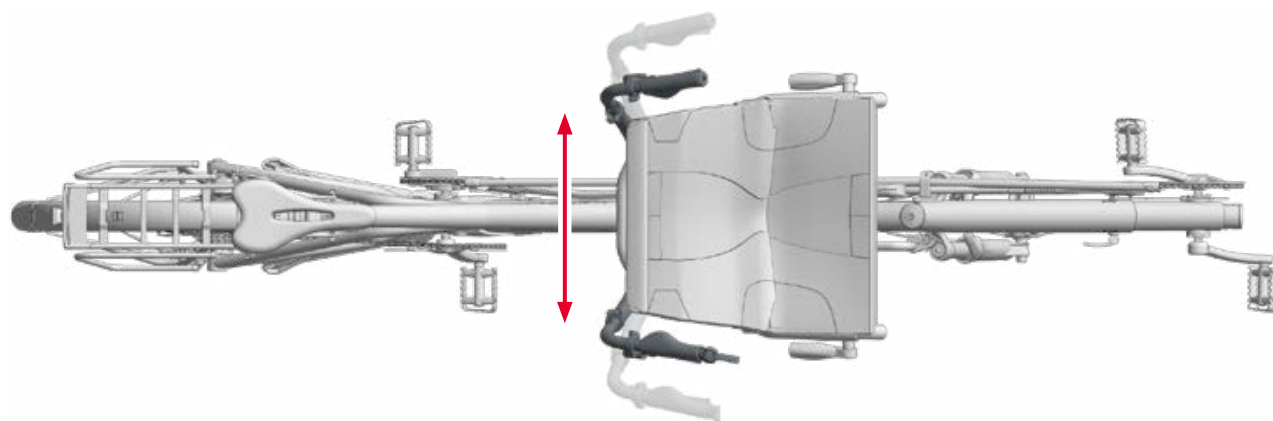
HANDLEBAR ADJUSTMENTS

The **angle** and **width** of the **handlebars** can be adjusted in order to find an ergonomic sitting position. Stops can be used for remembering the position of the handlebar after folding. See page 13.

The **height** and **angle** of the **handlebar grips** can be adjusted. See page 14.

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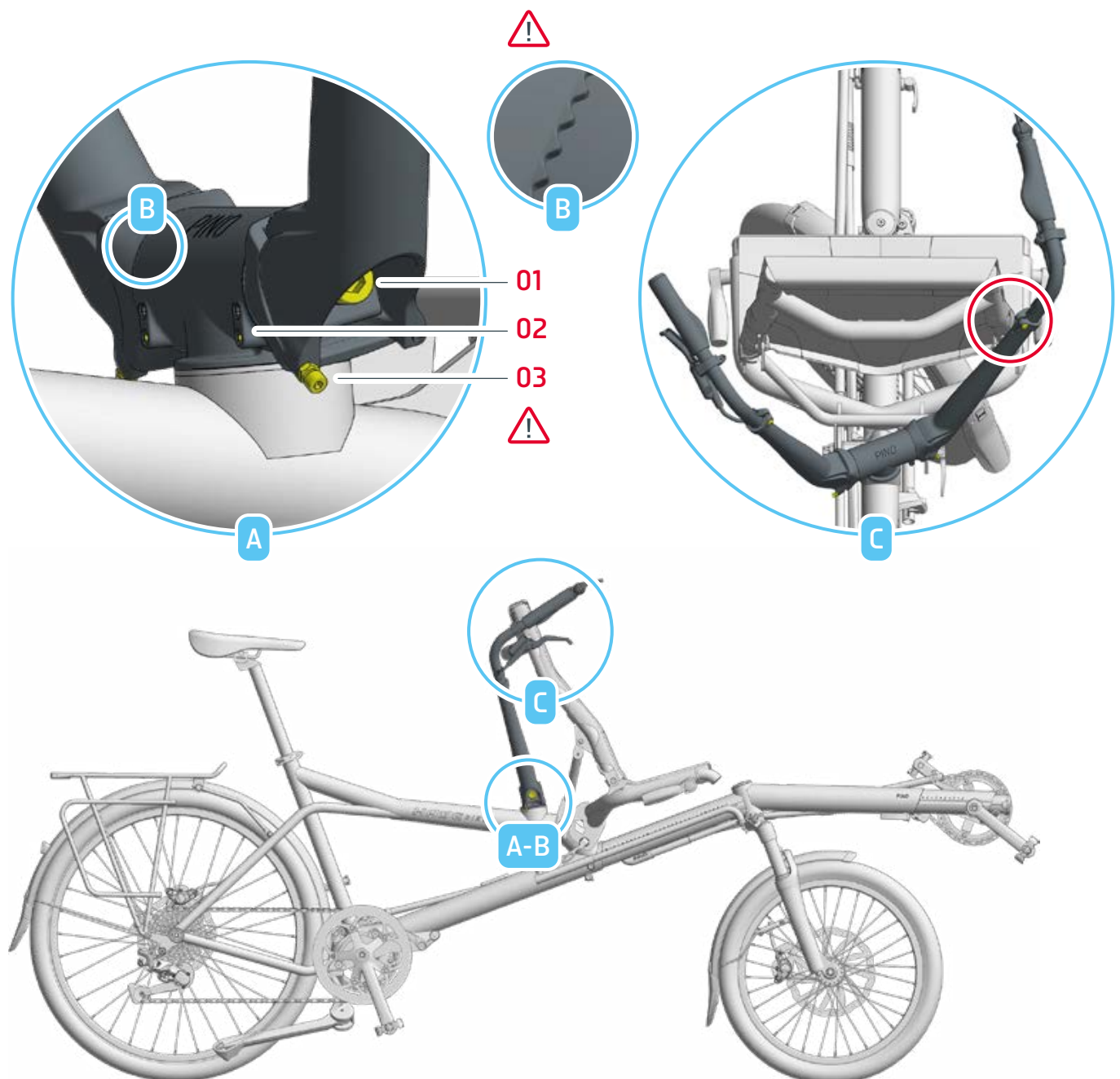
ADJUSTING THE HANDLEBARS

1. Loosen the bolt **(01)**.
2. Tilt the handlebars forward or backward and
3. inward or outward to the desired position.
4. Tighten the bolt **(01)** with a torque of 46 Nm [34 ft-lb].

! CAUTION! Make sure that the joints are correctly interlocked. **(B)**

5. Check that the handlebars can be moved freely in both directions without restriction. **(C)**
6. Use the limit screws on both sides **(02)** and **(03)** **(A)** to remember the position of the handlebars after folding.

! CAUTION! Do not tighten the limit screws **(02)** and **(03)** too firmly.

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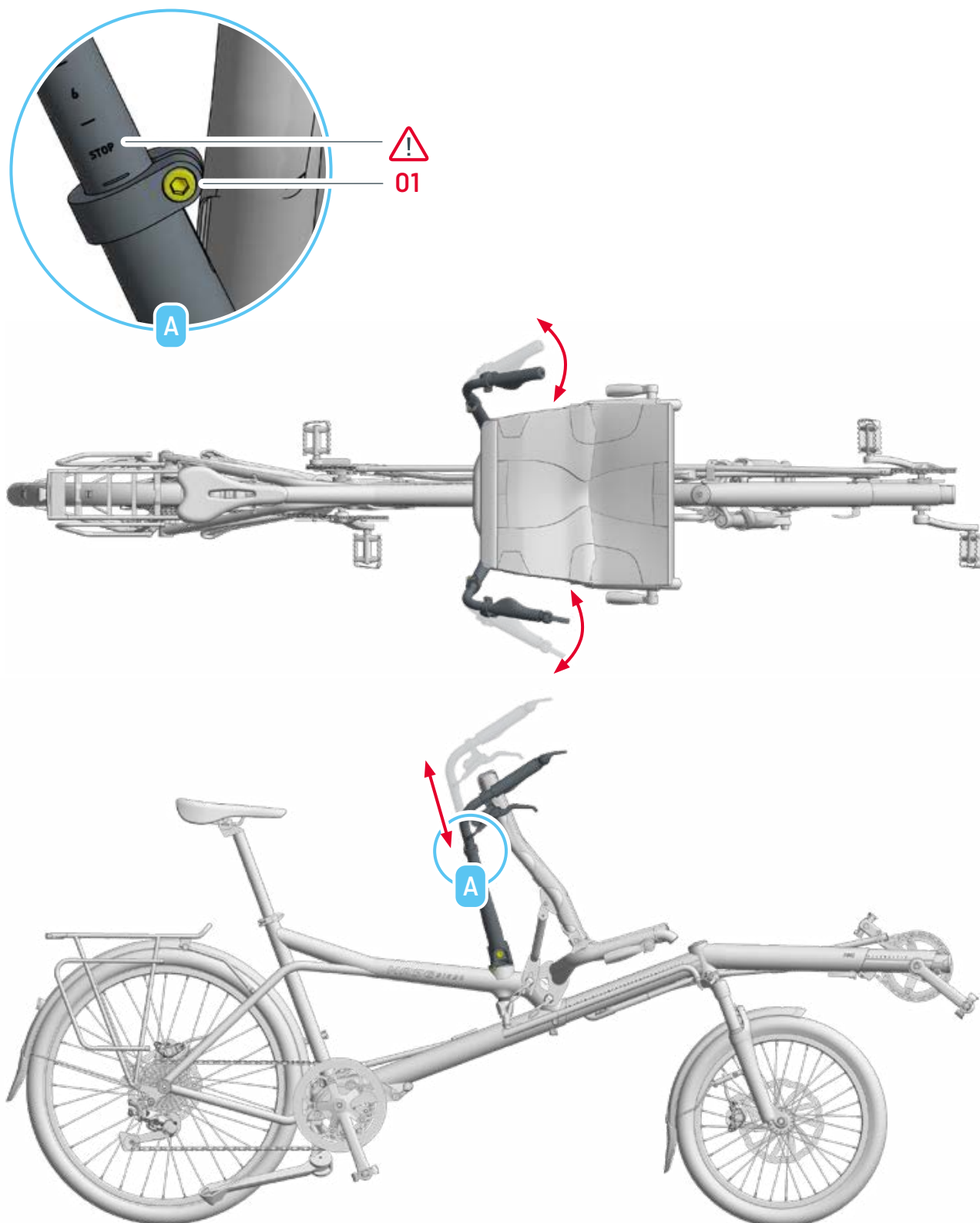
ADJUSTING THE HANDLEBAR GRIPS

1. Loosen the bolt **(01)**.
2. Adjust the **height** and **angle** of the handlebar grips.
3. Retighten the bolt **(01)**.

 **CAUTION!** Never ride your bike with the handlebar grips drawn out beyond the "STOP" mark.

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CHECKING THE SEAT POSITION OF THE FRONT RIDER

Sit in the front seat of the PINO with your feet on the pedals and the balls of your feet over the pedal axles. When one of the pedals is in front and below the level of the front boom (as shown in the illustration), the lower leg should be parallel to the crank arms.

PLEASE NOTE! Consult your authorized dealer for assistance.

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
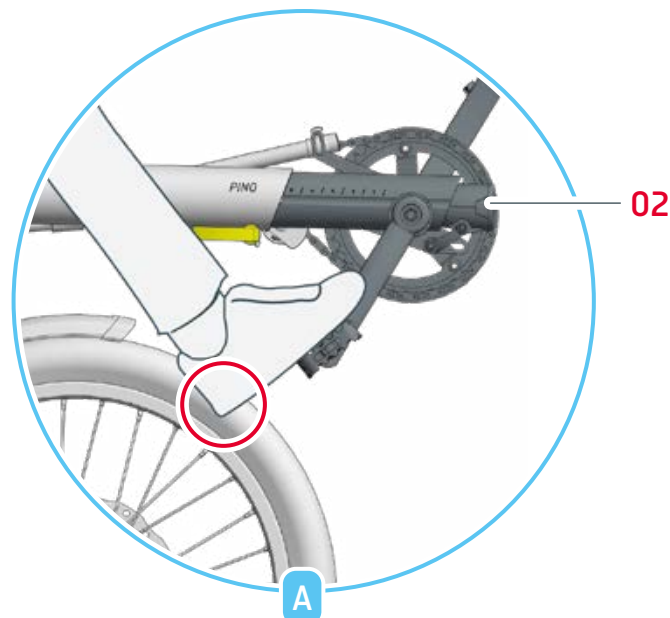
ADJUSTING THE FRAME LENGTH USING THE FRONT BOOM

1. Open the quick-release lever (01) on the middle frame.
2. Push the front boom (02) further into the frame or pull the front boom (02) further out of the frame.

! CAUTION! Never ride your bike with the front boom drawn out beyond the "STOP" mark (B).

3. Retighten the quick-release mechanism (01).
4. Check to make sure that there is sufficient space between the front rider's heel and the front wheel (A).

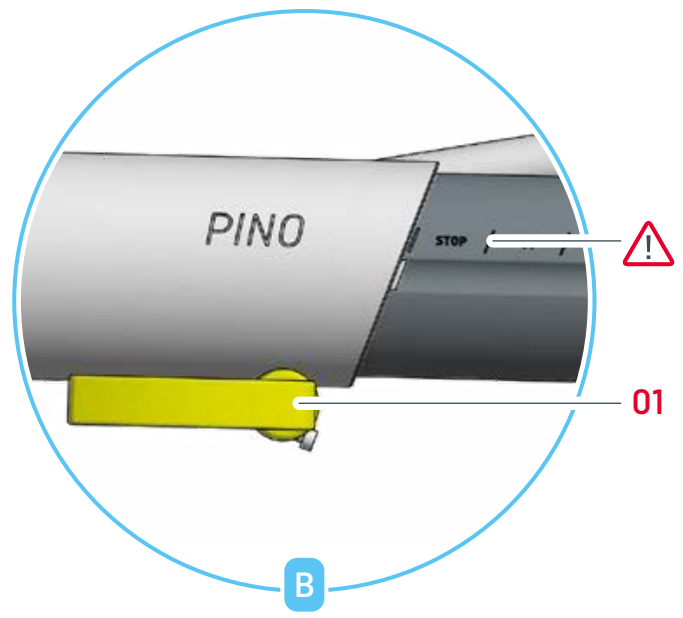i PLEASE NOTE! If the heel comes in contact with the front wheel when pedaling, the frame length must also be adjusted using the middle frame. See page 17

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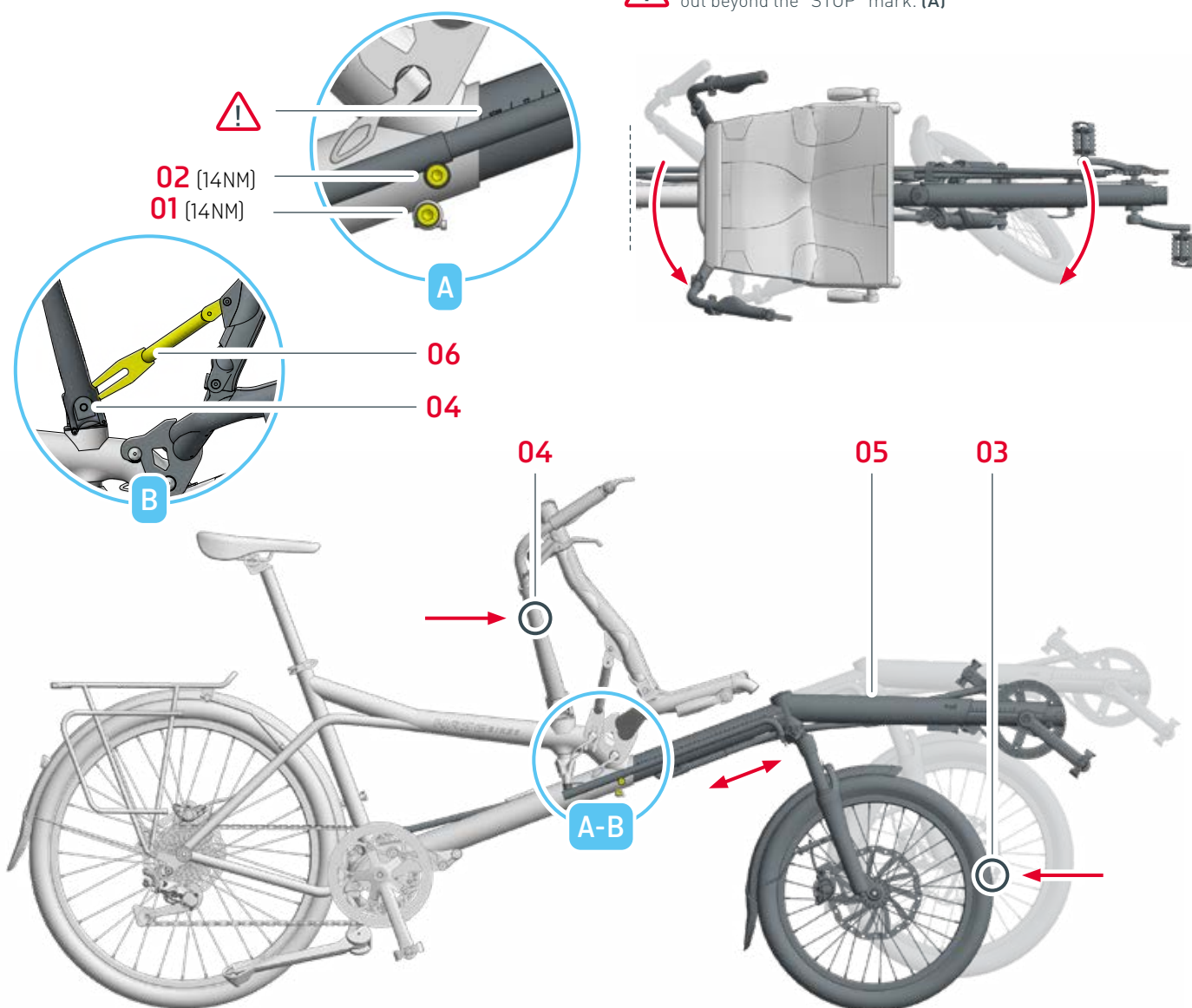
ADJUSTING THE FRAME LENGTH USING THE MIDDLE FRAME

1. Loosen the bolt **(01)** on the main frame.

! CAUTION! Never loosen the bolt **(01)** by more than 4 full turns.

2. Standing on the right side of the PINO, hold the front wheel **(03)** in one hand and the handlebar grip **(04)** in the other.
3. To lengthen the frame, pull the front of the wheel and the handlebar grip towards one another (see upper right illustration), and to shorten the frame, push the front of the wheel and the handlebar grip further apart.
4. Loosen the bolt **(02)**, and position the handlebars perpendicular to the front wheel. Use the seat stay **(06)** as reference by laying it against the handlebar **(04)**, this way the handle bar will be set in the right position while aligning the front wheel.
5. Tighten the screws **(01)** and **(02)** with a torque of 14Nm (10.33 ft-lbs) each.

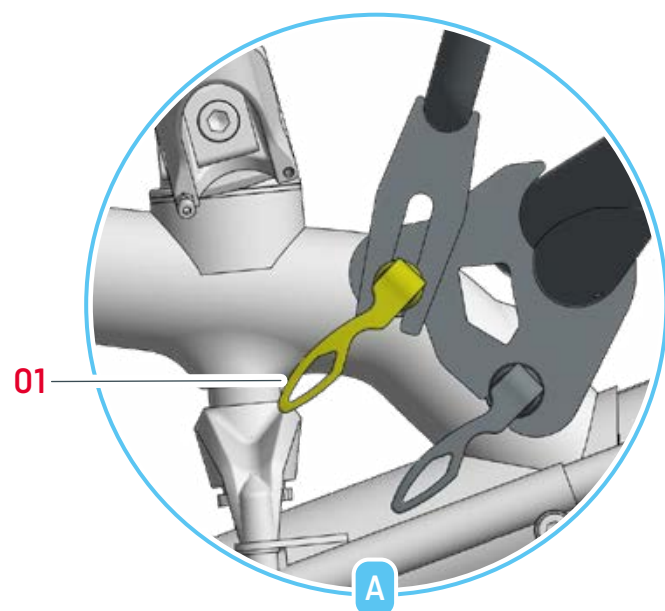
! CAUTION! Never ride your bike with the middle frame **(05)** drawn out beyond the "STOP" mark. **(A)**

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

ADJUSTING THE SEAT ANGLE

1. Open the top quick-release lever on the seat stay (01).
2. Tilt the seatback into the desired position.
3. Retighten the quick-release mechanism (01).
4. Check that the handlebars can be moved freely in both directions without restriction. (B).

PLEASE NOTE! If the handlebars hit against the seatback, then the width or angle of the handlebars must be readjusted.

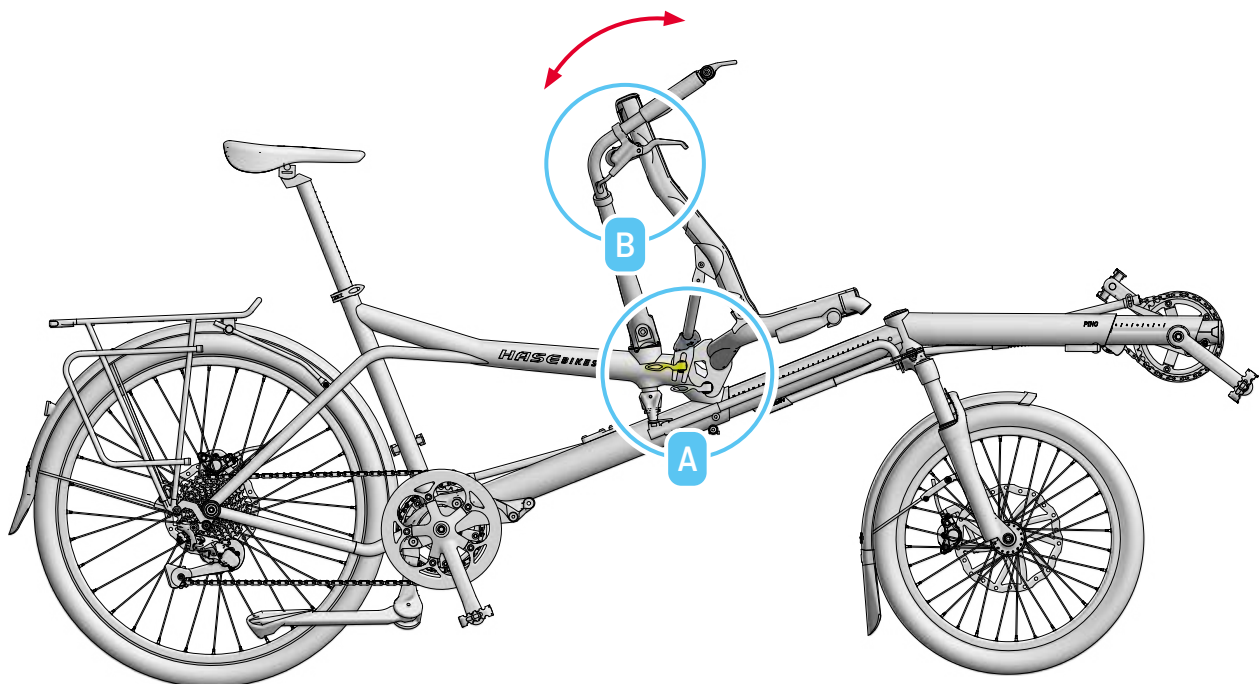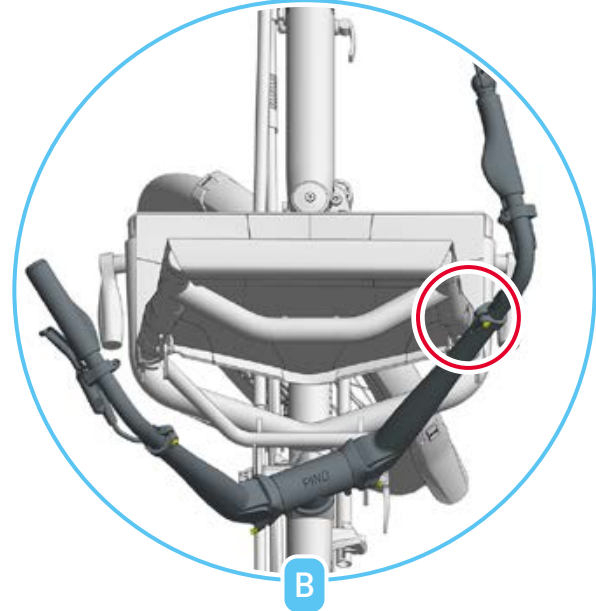CAUTION! The steering must not be blocked by the seat back-rest.

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CORRECT TIRE PRESSURE

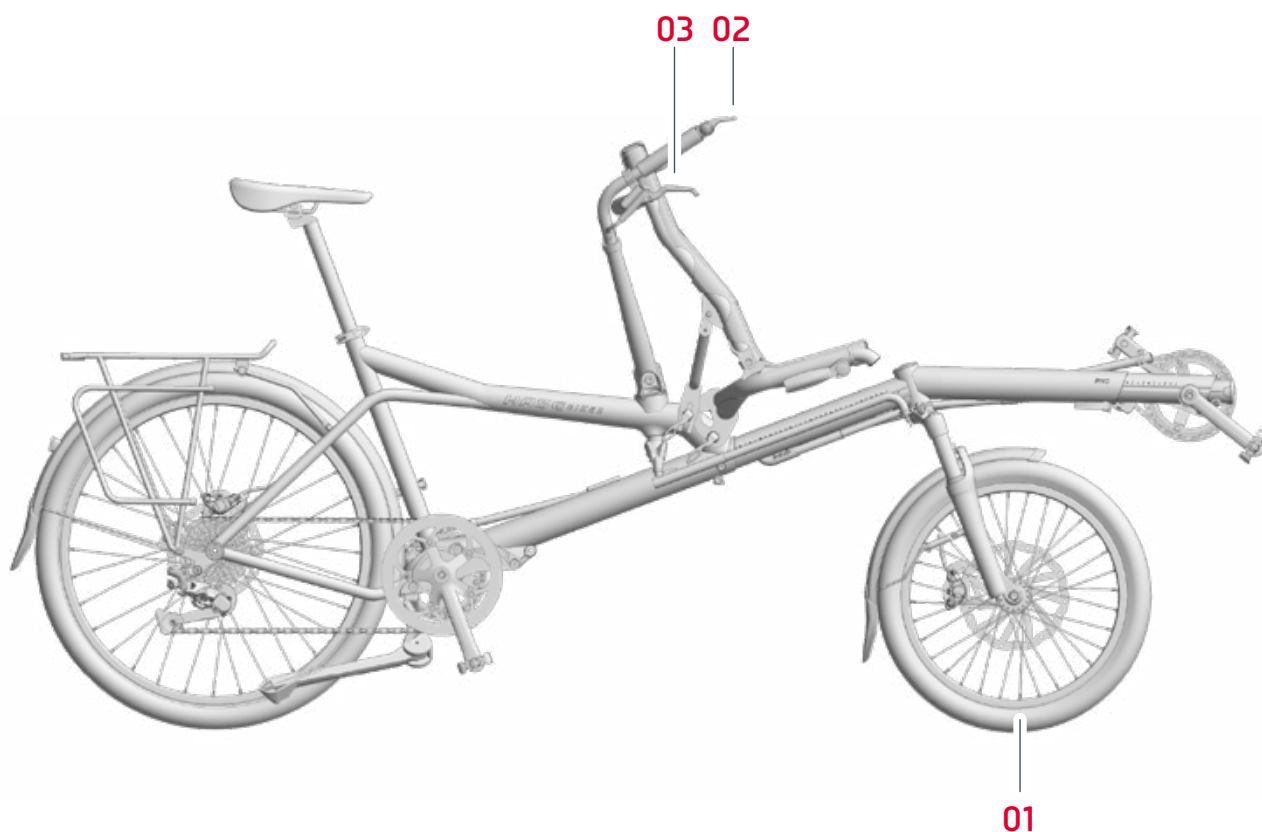
The PINO should only be ridden with the correct tire pressure. The permissible pressure range is marked on the tire sidewalls **(01)**.

SHIFTING GEARS

Before riding the PINO on public roads, be sure to familiarize yourself with the gear system **(02)** in an open area away from traffic. Shifting gears takes practice, which could distract you from the road and reduce your awareness of others.

BRAKING

Your PINO is equipped with particularly powerful hydraulic brakes **(03)**. The right brake lever controls the rear-wheel brake, and the left brake lever controls the front-wheel brake.

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CARRYING CARGO

The rear rack can hold two panniers **(01)**.

! CAUTION! The braking distance of the PINO can be increased by heavy loads. Before your first ride with cargo, be sure to familiarize yourself with the braking and handling characteristics in an open area away from motorized vehicles.

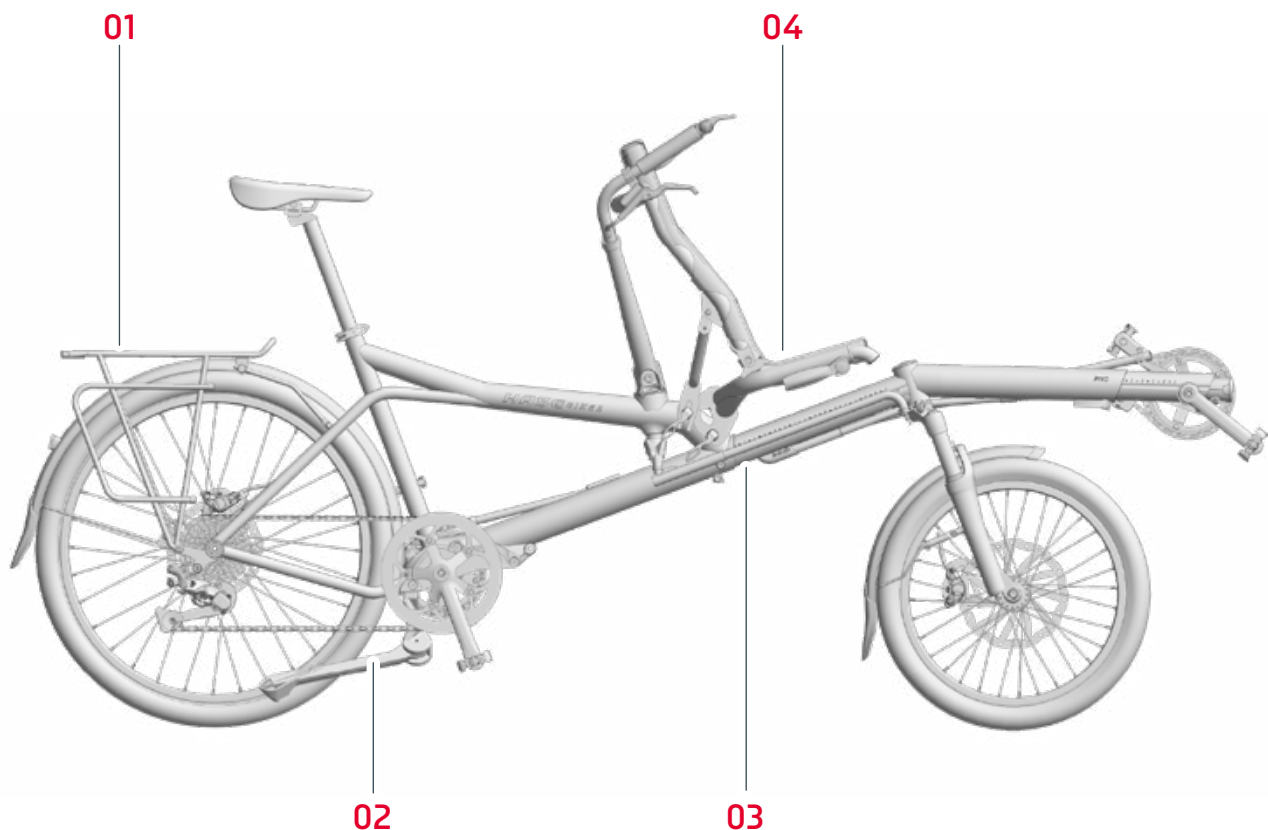
PARKING THE PINO

The PINO can be parked securely using the HASE Double Kickstand **(02)** available as an accessory. The kickstand is designed to support the weight of the luggage and riders when the tandem is parked. Equipping your PINO with a HASE Double Kickstand **(02)** is particularly advisable when the front rider is a child. With the PINO parked on the Double Kickstand, you can easily buckle your child into the seat with no danger of tipping.

! CAUTION! Usable frame length: adjustment range **6 – Stop** on the middle frame scale **(03)**.


RIDING WITH CHILDREN

The recumbent position of the front seat **(04)** is ideal for children. If the child's feet do not reach the pedals at the shortest frame setting, short crank arms must be used. With these crank arms, children as small as 1 meter (3' 3") in height can pedal in the front seat. An additional actively pedaling child can be brought along by hitching a TRETTS trailer to the PINO.

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

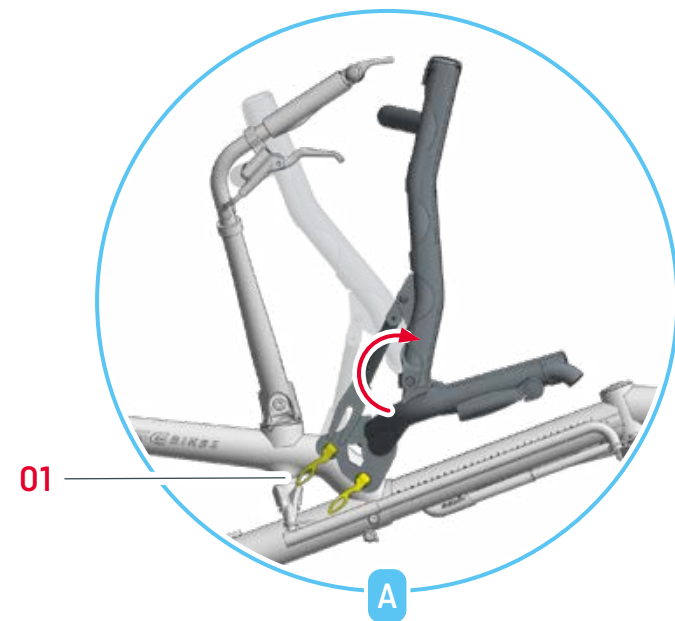
REMOVING THE SEAT

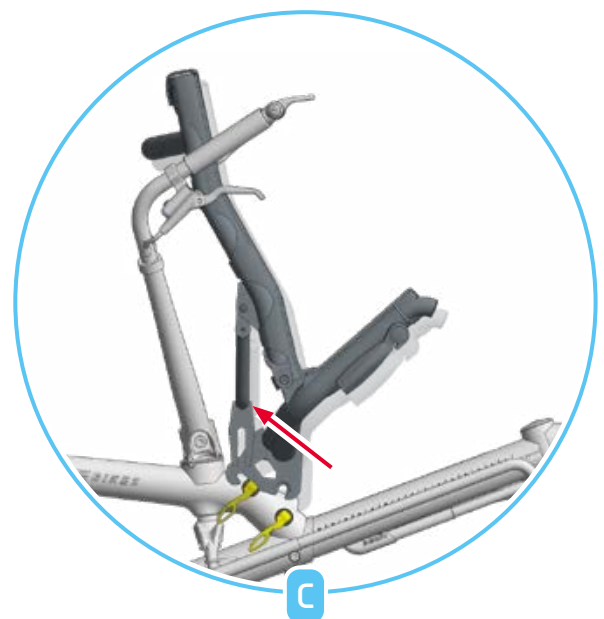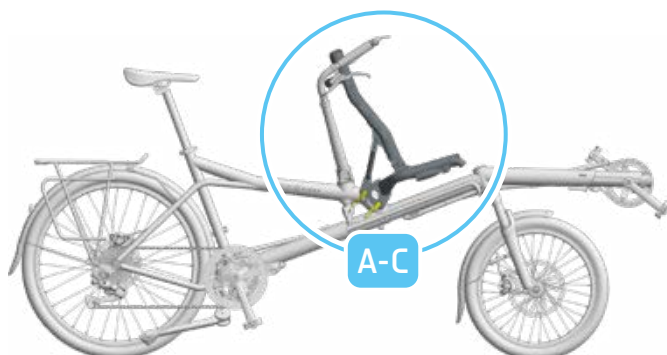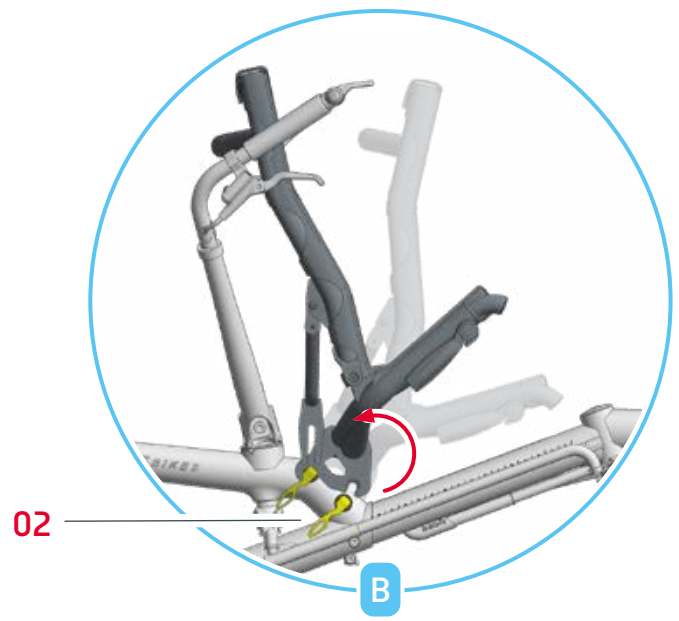1. Open the upper quick-release **(01)**.
2. Open the lower quick-release **(02)** and loosen the nut three full turns **(B)**.
3. Tip the seatback forward **(A)**.
4. Then tip the entire seat upward, pivoting on the upper quick-release **(01)** **(B)**.
5. Lift the whole seat up and back, towards the handlebars **(C)**. You can now remove the seat.

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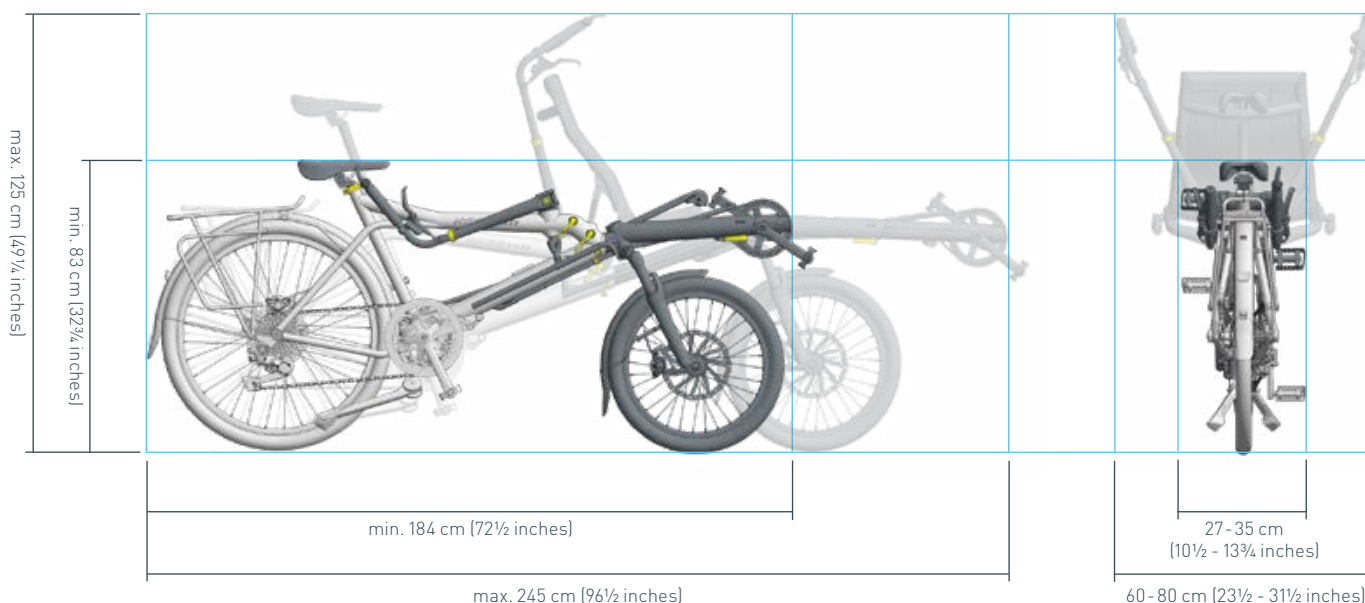
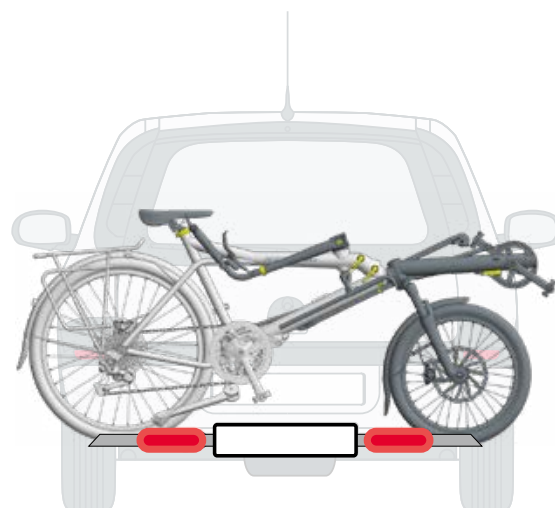
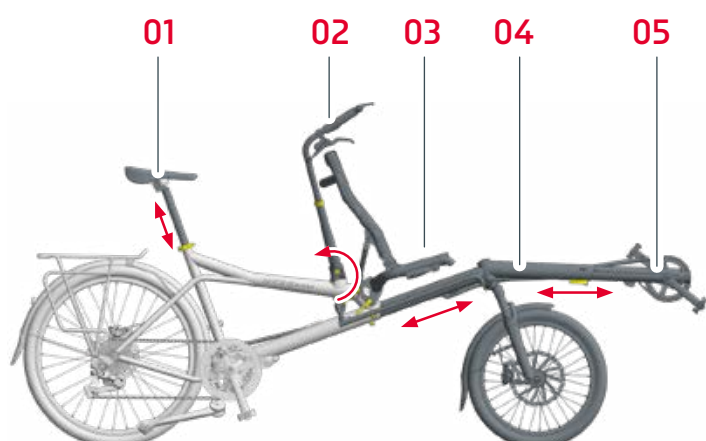
TRANSPORTING THE PINO

1. Push the seatpost **(01)** all the way into the main frame.
2. Tip the handlebars **(02)** back, toward the saddle.
3. Remove the front seat **(03)**.
4. Push the middle frame **(04)** as far as it will go into the main frame.
5. Push the front boom **(05)** as far as it will go into the middle frame.

PLEASE NOTE! You can find more detailed instructions for the individual steps on pages 8-17 and 21.

PLEASE NOTE! The dimensions (see below) can vary owing to the use of optional parts like seatposts or the front chainring.

PLEASE NOTE! When reduced to its shortest frame length, the PINO can be transported on standard bike carriers. When carried on a rear-mounted car rack, make sure that the PINO does not protrude beyond the width of your vehicle. The regulations for the transport of bicycles on rear-mounted car racks vary from country to country.

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
PINO

Total length	184 - 245 cm (72½ - 96½ inches)
Total width	60 - 80 cm (23½ - 31½ inches)
Total height	83 - 125 cm (32¾ - 49¼ inches)
Weight	from 19.2 kg (42 lb)
Permissible load	max. 225 kg (496 lb)
Brake system	hydraulic disc brakes, tandem version
Gear system	11-speed Shimano derailleur
Gear system option	14-speed Rohloff Speedhub
Chain length	front max. 4150 mm (164 links), rear standard
Frame	6061 aluminum
Wheels	26" rear, 20" front
Rims	double wall aluminum with eyelets
Tires	rear 50-559, front 47-406
Height of riders	rear: 1.5 - 2 m (4'11" to 6'7") , front: 1.0 - 2 m (3'3" to 6'7")

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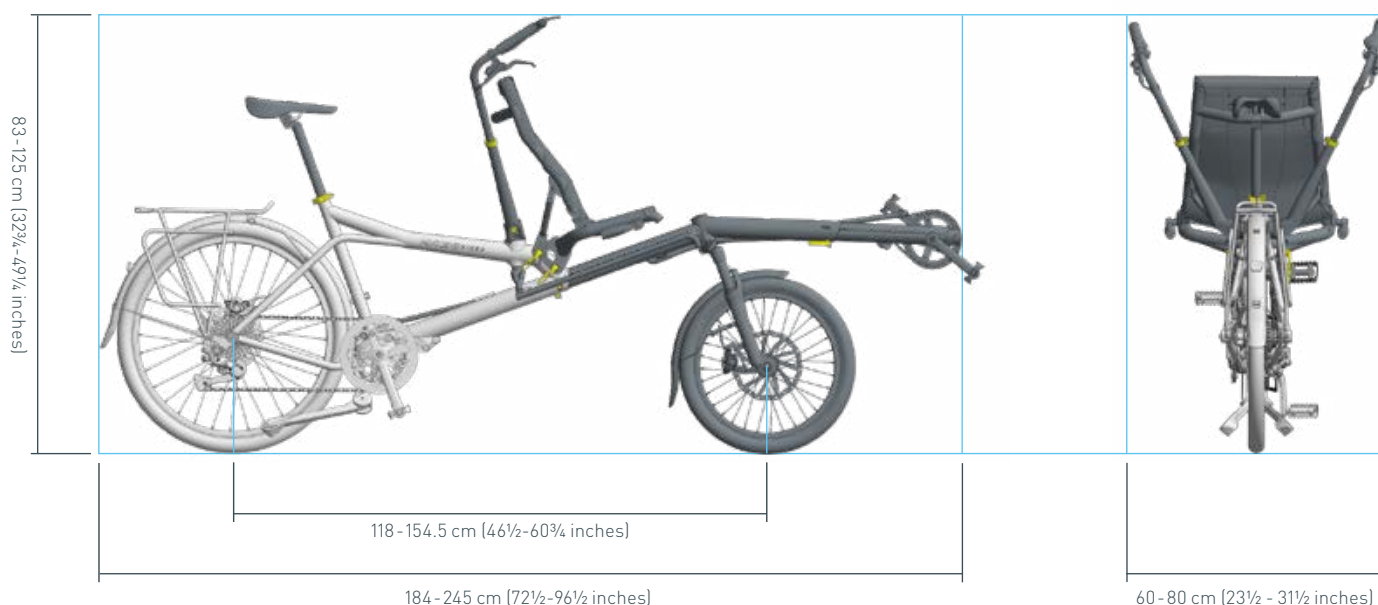SERVICE INTERVALS

i PLEASE NOTE! All inspection and maintenance work should be performed by an authorized HASE BIKES dealer.

All PINOs require a first service after 300 km (200 miles). Subsequent servicing should be carried out every 2000 km (1250 miles) or 12 months. See also page 30, "Service log"

Component	Action	Before Each Ride	Once a Month	Once a Year
Lighting system	Check for proper function	X		
Tires	Check tire pressure	X		
	Check tread wear and sidewalls	X		
Brakes	Perform a brake test	X		
	Check pad thickness		X	
	Check alignment to the brake disc		X	
Brake hoses	Check for damage or cracks	X		
Bolts of the frame-telescoping mechanism	Check for tightness	X		
Chain	Check for wear		X	
	Lubricate		X	
Crank bracket	Check bolts			X
Wheels	Check for lateral/radial true		X	
	Check spoke tension		X	
Handlebars	Check			X
Steering rod	Check for play			X
Headset	Check for play			X
Hubs	Check for play			X
Pedals	Check for play			X
Rear derailleur	Clean and lubricate		X	
Gear cables	Check for damage or cracks		X	
Quick-release mechanisms	Check tightness	X		
Nuts and bolts	Check tightness		X	
Bottom bracket and front crankset	Check for play			X

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CHECKING WHEELS FOR LATERAL/RADIAL TRUE

During the running-in period, stress on the wheels may cause the spokes to settle in the rims so that the spoke tension is no longer uniform. As a result, the wheels can become untrue. Have your wheels **(01)** re-trued by your authorized dealer at the first service after approx. 300 km (200 miles). Subsequent tensioning is seldom necessary.

CHECKING THE TIRES

Check the tires for damage and tread wear. Tires with cuts, cracks, or worn tread should be replaced. Check the tire pressure before each ride, and always ride with the pressure specified on the tire sidewalls.

CHECKING THE DISC BRAKES

Please read the information provided in the enclosed manual from the brake manufacturer. Have the thickness of your brake pads **(02)** checked regularly by a qualified bicycle mechanic. If you notice any unusual braking sounds or inadequate braking strength, consult your bicycle mechanic immediately.

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CHECKING AND ADJUSTING THE DERAILLEUR

Please read the information provided in the enclosed manual from the gear system manufacturer. The derailleur **(01)** of a new PINO has been carefully adjusted for smooth shifting. Over time, however, the gear cables tend to stretch, and the derailleur may no longer function with the same precision.

CHECKING THE CHAIN TUBES

The purpose of the chain tubes **(02)** is to protect clothing from chain lubricants. Over time, friction from the moving chain will cause the tubes to wear. Check the ends of the chain tubes for signs of wear. If the ends are completely worn, have your authorized dealer replace the tubes.

CHECKING THE CHAIN

The PINO uses two chains. The front chain **(03)** is longer than that of a standard bicycle and has no gear system. This chain does not wear as quickly as a normal chain and therefore does not need to be replaced as often. The rear chain **(04)** is shorter and has a gear system. Therefore, this chain wears more quickly than the front chain. You can check the condition of the chain using a chain wear gauge from a bike shop. We recommend that you have a qualified bicycle mechanic regularly check the condition of the chain and, if necessary, replace the chain, chainrings, and/or cassette. To accommodate the maximum frame length of the PINO, the front chain must be 4150 mm in length. This is equivalent to 164 chain links. The rear chain has a standard length.

CLEANING AND OILING THE CHAIN

In order to ensure that the chain does not wear out before the end of its maximum service life, it must be cleaned and lubricated on a regular basis.

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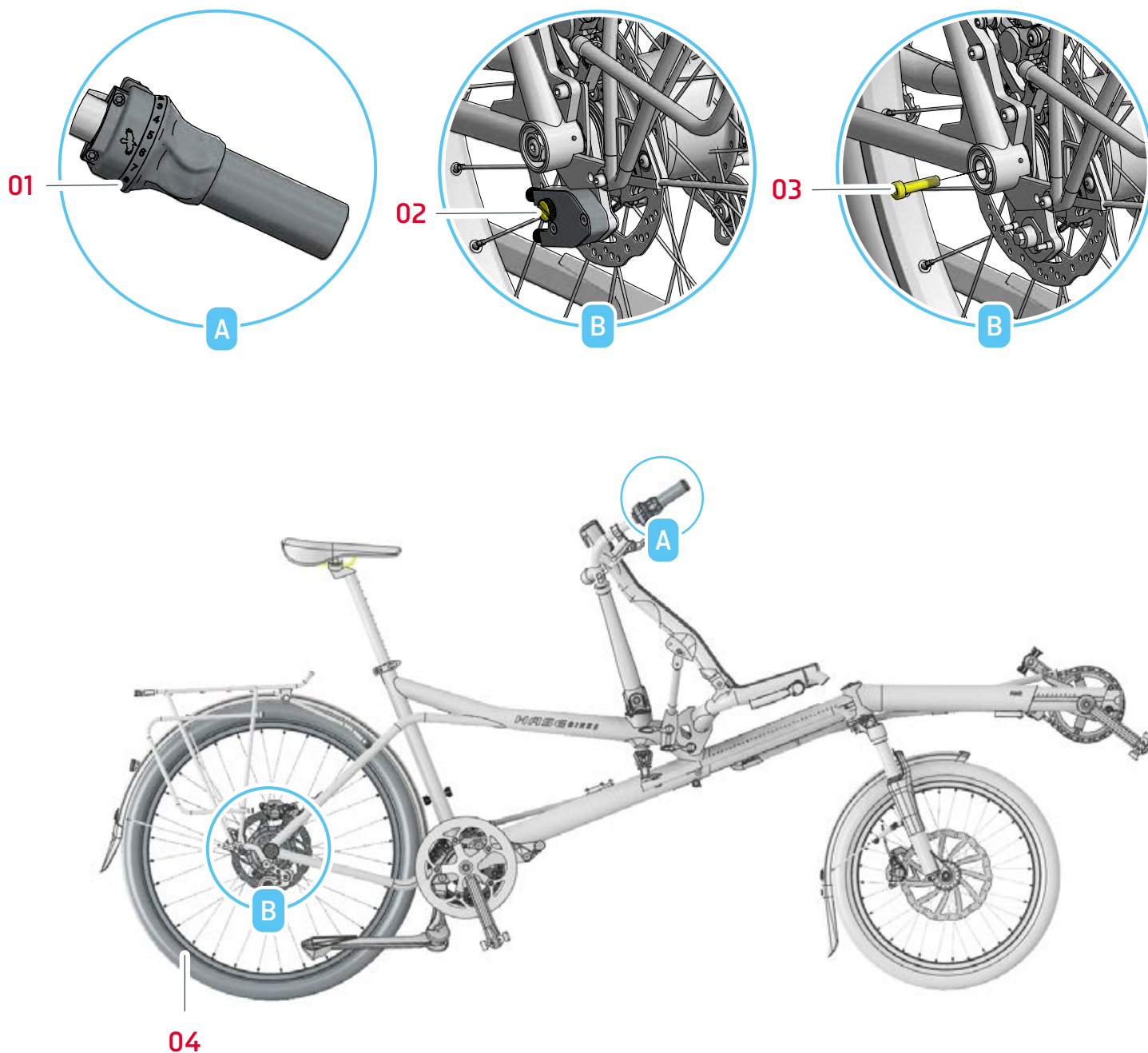
ROHLOFF SPEEDHUB REAR WHEEL DISASSEMBLY

1. Turn the shifting handle **(01)** to the highest or lowest gear first.
2. Loosen the thumbscrew **(02)** of the switch box and disassemble it.
3. Loosen the two M7 x 30 screws **(03)** on both sides.
4. The rear wheel **(04)** can be removed downwards.
5. For subsequent assembly, proceed in reverse order.

NOTE! When installing and removing the wheel, pay attention to the fit of the brake caliper and brake disc.

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The statutory warranty applies. The dealer must sell you the HASE BIKE in a safe and roadworthy condition. In addition, the company HASE BIKES offers a warranty of three years (from the date of purchase) against frame breakage in accordance with the following conditions. By registering your bike at <https://hasebikes.com/209-1-Bicycle-passport.html> you can extend this warranty to 5 years.

- The company HASE BIKES is only liable for defects caused by faulty workmanship on the frame or in the assembly process. Components are covered by the respective manufacturer's warranty.
- The additional warranty from HASE BIKES only applies to the original owner and does not cover damage resulting from any use, changes, or repairs that are not in accordance with the user's manual.
- Within the warranty period of 3 years from the date of purchase, HASE BIKES will repair any broken frame part free of charge. If the part cannot be repaired, it will be replaced free of charge with a new or like-new part. Any defective parts that are replaced by the company HASE BIKES become the property of HASE BIKES.
- Any further claims beyond the scope of this warranty are excluded. In particular, the company HASE BIKES will not cover the costs of removing or installing parts (e.g. by the dealer) or shipping costs. We do not accept postal deliveries with insufficient postage.
- Never exceed the maximum permissible load (rider and cargo) of 225kg (496 lb) specified for the PINO.
- The statutory and HASE BIKES warranty will immediately become void if the PINO is used for competition purposes, ridden on rough terrain, or used in any manner considered abnormal for a bicycle.
- Damage resulting from accidents is not covered by the warranty.
- The warranty also becomes void if the PINO is fitted with incompatible accessories or if accessories are installed incorrectly. Therefore, if you need to replace or install any components, be sure to have the associated work performed by an authorized dealer.
- The warranty becomes void if any improper modifications (e.g. grinding, drilling, bending, etc.) are made to the frame.
- Any repairs or replacements made without charge during the warranty period do not carry a new warranty beyond that of the original purchase.
- A warranty claim can only be processed through the authorized HASE BIKES dealer from whom the bike was purchased or through one of our distribution partners.
- Children may only use the PINO under parental supervision.

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Datum

SERVICE LOG

Have your bike checked by your authorized dealer on a regular basis. In order to avoid unnecessary downtimes and waiting periods, it is advisable to have the service performed during the autumn or winter months, or make an appointment in advance during the high season.

Special features

.....

.....

.....

.....

.....

.....

Frame no.

Frame color

.....

CHECKLIST

- Check pedals for tightness
- Check headset
- Check handlebars for tightness
- Check front and rear brakes
- Check front and rear wheel bearings
- Check bottom bracket and front crankset
- Check crank arms for tightness
- Check saddle for tightness
- Check chain tension and wear
- Check tires and air pressure
- Check rims and spokes
- Check all screws for tightness
- Check gear system
- Check lighting system
- Check accessories, e.g. check motor system

1ST SERVICE

300 km (200 miles) or three months after date of purchase.

Replaced or added parts

.....

.....

Stamp/signature of dealer

Job no.

Date

.....

2ND SERVICE

2000 km (1250 miles) or one year after date of purchase.

Replaced or added parts

.....

.....

Stamp/signature of dealer

Job no.

Date

.....

3RD SERVICE

4000 km (2500 miles) or two years after
date of purchase.

Replaced or added parts

Stamp/signature of dealer

Job no.

Date

4TH SERVICE

6000 km (3750 miles) or three years after
date of purchase.

Replaced or added parts

Stamp/signature of dealer

Job no.

Date

5TH SERVICE

8000 km (5000 miles) or four years after
date of purchase.

Replaced or added parts

Stamp/signature of dealer

Job no.

Date

6TH SERVICE

10,000 km (6250 miles) or five years after
date of purchase.

Replaced or added parts

Stamp/signature of dealer

Job no.

Date

HASE BIKES
Hiberniastraße 2
45731 Waltrop
Germany

Phone +49 (0) 23 09 / 93 77-0
Fax +49 (0) 23 09 / 93 77-201
info@hasebikes.com
www.hasebikes.com